

# BRUDYAL®

Wetting solution



300106

## INSTRUCTIONS FOR USE OF BRUDYAL WETTING SOLUTION

### APPLICATIONS

**BRUDYAL Wetting Solution** are wetting drops that relieve the symptoms of irritation and dry and tired eyes. Its special formulation contains Sodium Hyaluronate, a natural substance present in the eye that forms a uniform and protective film with great water absorption capacity and resistance to evaporation. It delivers a high degree of moisturisation to the eye, thus improving visual comfort and the wearing of contact lenses. It is particularly recommended for contact lens users in situations that increase dry or irritated eyes.

### INSTRUCTIONS FOR USE

- Wash your hands with water and soap before using the product.
- Take an ampoule of **BRUDYAL Wetting Solution** and twist the upper tab to open it.
- Direct use: Instil one or two drops in each eye and blink several times.
- Use in contact lens: Apply one or two drops to the inner side of the lens before inserting it in your eye.
- Use **BRUDYAL Wetting Solution** as often as necessary.

## PRECAUTIONS

- Keep out of the reach and sight of children.
- This product does not contain preservatives. After opening, the ampoule should be used immediately and any surplus solution subsequently disposed of, as it could become contaminated and cause an infection.
- An ampoule may only be used by the same person.
- Do not use after the expiry date.
- Do not use if the ampoule is damaged.
- Do not use the product if you are allergic to any of its ingredients.
- In the event of eye irritation consult a specialist.
- Do not store above 25°C.

## COMPOSITION

Sodium Chloride, Boric Acid/Sodium Borate Buffer with 0.1% Edetate Disodium and 0.1% Hyaluronic Acid.



## MANUFACTURER

Unolab Manufacturing, S.L. Av/ De las flores, 6. 28970 Humanes de Madrid (Madrid). Spain

## DISTRIBUTED

**BRUDYLAB, S.L.** C/ Riera de Sant Miquel, 3 2º 4ª. 08006 Barcelona. Tel. +34 93 217 03 66